-  *Bedienungsanleitung*
-  *Instruction Manual*
-  *Manual de instrucciones*
-  *Mode d'emploi*

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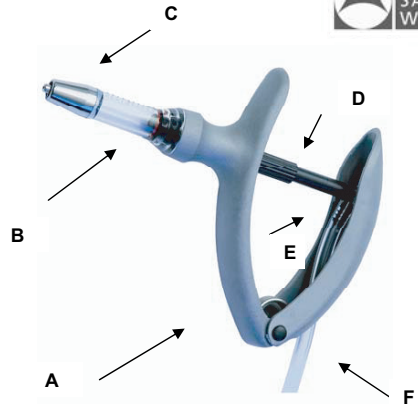
ECO – MATIC®
by **HSW**

- Automatic self-filling syringe for injections
- with tubing attachment

- A) ergonomic designed handle
- B) plastic barrel with piston and O-ring
- C) metal needle fitting - Luer Lock
- D) dosage adjustment
- E) tube inlet socket
- F) tube insert holde in the handle

Instruction for use

The HSW ECO-MATIC® self-filling dosing syringe is designed for continuous use, only the O-ring and the barrel can be changed if necessary.

It is essential that the syringe is cleaned before each use. Please follow the manufacturers' recommendations with any cleaning or disinfectant solutions used.

1) TO FIT THE NEEDLE

Attention:

- For your safety: Leave the protective cover (only on disposable needles) on the needle until ready for use.
- Insert needle onto Luer Lock fitting and secure the needle clockwise.
- Disconnect the needle twisting the opposite way.

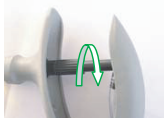
2) ATTACH DRAW – HOSE INTO THE INLET SOCKET

1. Insert draw hose into the tube inlet socket of the syringe.
2. Push the end of the draw hose through tube insert hole on the handle and straighten the tube.

3) SET DOSE

- Turn the dose adjuster either clockwise or counter-clockwise until front of the piston is aligned with the dose setting printed on the barrel.
- In some syringe models the dosage is fixed at a certain volume and cannot be adjusted manually.

Dose setting

4) FILLING OF THE ECO-MATIC®

1. Remove protection cover on the needle.
2. Hold ECO-MATIC® at a slight incline.
3. Squeeze adjustable handle and then release quickly to draw fluid into barrel.
4. To remove air bubble inside barrel, repeat step (c) taking care not to waste fluid.
5. To fill the syringe and prepare for injection please make sure that the syringe is adjusted on volume, 0.5 ml (for the 1 ml fine dosage syringe) to prime and that no air bubbles are visible. Hold the syringe upright and squeeze the plunger so that air bubbles can be released. From now on any dosage will be possible.

INSTRUCTION AFTER USE OF THE ECO-MATIC®

Remove all traces of material from syringe by flushing thoroughly with a hot water detergent mix. Rinse thoroughly with clean, warm water.

Attention! If you detect that the syringe draws air after the described cleaning, it is possible that a valve is fixed in the front metal approach. The valve can be released by knocking slightly the front of the syringe against an object, e.g., the edge of a table, and afterwards rinsing the syringe with warm water.

DISINFECTION

It is recommended that the ECO-MATIC® and a supply of needles or nozzles be disinfected before and after each use.

1. Suck "hot" water into the barrel by using an attached bottle with warm water.
2. Suspend the complete ECO-MATIC® including needles or nozzles in a container of water and boil for up to 20 minutes. Ideally, the cylinder will be disassembled from the handle and the syringe will be boiled in a disassembled condition.
3. Remove the ECO-MATIC® from container, wrap cloth around the handle and pump it dry.

TO LUBRICATE

Remove plastic barrel from the handle by turning it. Lubricate piston and glass barrel with 100% pure oil, cold squeezed or equivalent vegetable oil.

Please follow the manufacturers durability and storage life instruction.

Do not use mineral oil, silicone, WD-40, glycerine or alcohol. Petroleum based products harden the piston O-ring, reducing the life of the component.

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© copyright - Henke-Sass, Wolf GmbH - 740207-03/19 · Änderungen in Technik und Design vorbehalten · Printed in Germany
© copyright - Henke-Sass, Wolf GmbH - 740207-03/19 · Subject to amendments in technology and design · Printed in Germany
© copyright - Henke-Sass, Wolf GmbH - 740207-03/19 · Reservado los derechos de modificaciones técnicas y de diseño · Impreso en Alemania
© copyright - Henke-Sass, Wolf GmbH - 740207-03/19 · Sous réserve de modifications dans le design et la technique · Imprimé en Allemagne